



cologuard[®]

HOW TO RETURN YOUR KIT

USE THE FOLLOWING INSTRUCTIONS TO RETURN YOUR COLLECTION KIT TO THE LAB VIA UPS.

If you need assistance, please call us at 1-844-870-8870. We are available 24 hours a day, 7 days a week to answer your questions.

**EXACT
SCIENCES
LABORATORIES**

www.cologuard.com
Rx Only

LBL-10034rev7

Cologuard[®] colorectal cancer screening test is a registered trademark of Exact Sciences Corporation.

PREPARE YOUR KIT FOR SHIPMENT



a.

a. Locate the UPS shipping label on top of your kit.


- Shipping labels on the kit may differ from image.



b.

b. If the UPS shipping label has a green tear strip, it must be removed.

- Lift the left corner of the tear strip and pull it off.
- Lift the bottom edge of the label and pull up.
- Tear the top label off the kit along the perforation.
- Your return label now shows.
- You are ready to ship your kit.
- Continue with the next step.

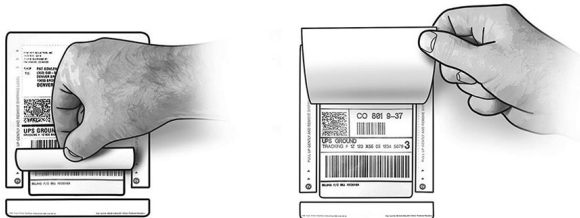

c. If the UPS shipping label has **no** green tear strip, you are ready to ship your kit.

RETURN YOUR KIT USING UPS

Plan to collect your sample when you can get it back to UPS that same day or the next day.

- Add a reminder to your calendar or phone on the day you will collect your sample.
- Make sure a Sunday or holiday will not delay your sample return.

WAYS TO RETURN YOUR KIT

Choose the no-cost return option that works best for you:

- Drop it off at UPS. Visit Cologuard.com/UPS to see your local options and hours. Remember, some places are closed on Sundays or holidays.
- Ask for a contact-free UPS pick-up. Call us at 1-844-870-8870 for help or visit Cologuard.com/UPS to schedule it on your own.

If you live in Puerto Rico or a US Territory or have questions about shipping your samples, contact us at 1-844-870-8870.

STORING YOUR SAMPLES BEFORE SHIPPING

To store the samples before you return the box:

- Store your boxed and sealed samples at room temperature.
- Keep the samples away from heat and direct sunlight.
- Keep the samples out of the reach of children.
- Keep the kit upright.

Once you have shipped your samples to the lab, you are finished!

Your healthcare provider will contact you with your result.